

REPLACING THE BATTERIES

The device will beep three times and the LCD screen will show the following picture when it is time for new batteries. This picture will be shown at the start of a treatment or during a treatment. The device will turn off after this picture is displayed.



Never change the batteries when the device is running. Wait until the device stops operating to change the batteries.

Note: Remove the plastic covering from the batteries before replacing them.

The picture below shows the correct way to replace the battery. When replacing the battery, you will see the correct way to put them in on the inside of the battery compartment. Use **ONLY** the batteries supplied with the device. These are located in the carrying case.



TROUBLESHOOTING INFORMATION

Display Picture	Action to be Taken
	<p>Magnetic Interference – announced by three beeps. The "X" will flash.</p> <ol style="list-style-type: none"> 1. If the device looks damaged, call Customer Services. 2. If not, try moving to another location in your house. Make sure you are not near metal objects. 3. If the picture is still displayed, replace the batteries with new ones. 4. If the device still shows the picture, call Customer Services.
	<p>Phone For Help – announced by three beeps. Call Customer Services on 0800 5870857</p>

DEVICE CARE AND STORAGE

- If needed, clean the surface only with a damp cloth.
- Always store the device in its carrying case in a cool, dry place. Never keep it where the temperature is less than -15°C or greater than 60°C as this could damage the device.
- Never store the device or any of its parts in a car in cold or hot weather.
- Always store the device in the carrying case provided, as it will protect the device when not in use. The case also stores the extra batteries.
- To avoid the risk of choking or strangulation, always store the device in its case out of the reach of small children.

USE WHILST TRAVELLING

You should use the device at the same time each day. The clock inside the device counts each day starting at 8:00am Greenwich Mean Time (GMT) during daylight saving, and will only allow one treatment every 24 hours. If you travel outside of your normal time zone, you should aim to treat at the same time you normally would at home. Do not take the device through airport x-ray machines because the x-rays may damage the device.



Bone Growth Stimulation Patient Guide



DJO | 1a Guildford Business Park | Guildford | Surrey
 GU2 8XG | UNITED KINGDOM
www.DJGlobal.co.uk

© 2018 DJO - 00-6672-EN - Rev A

PATIENT INFORMATION


Your doctor has asked you to use a Bone Growth Stimulator to help your broken bone heal. It is safe and easy to use and comfortable to wear.

WHAT IS BONE GROWTH STIMULATION ?

The Bone Growth Stimulation device will create a magnetic field that stimulates your broken bone to heal. You will not feel the magnetic field the device produces when using it.



DIRECTIONS FOR DAILY USE

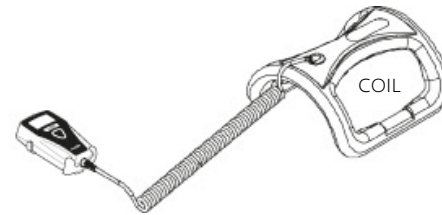


- Centre the device's coil around the broken bone.
- Close the strap, if you use one.
- To start a treatment, press the button on the front of the device, **hold it down until it beeps and then let go.**

- a. The treatment record will be displayed until the device beeps.
- b. The 30-minute treatment countdown will begin.



- c. After 30 minutes, the "treatment complete" icon will appear on the LCD screen, the device will beep twice and it will automatically shut off.

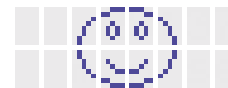

- Remove the device and store it until the next day (see device care and storage instructions).

DISPLAY SCREEN

When using the device, the LCD screen will show the time remaining for your daily treatment. In the example below, there are 29 minutes and 37 seconds remaining in the 30-minute treatment.



COMPLIANCE MONITORING RECORD

Your clinician may check your compliance record at any time. Pressing the button and letting go before it beeps will show the treatment. The LCD screen will look like this:



The first number on the screen is the number of treatments carried out (124 in the example above). The second number is the number of days since treatment was started (204 in above example).

If the daily treatment has been completed when the button is pressed, the treatment record will be shown followed by the "treatment complete" icon.