

**Software in Brief:** VitalStim® Plus PC Software (part number 13-5923-PC-SW) version 1.0.12, or latest, works with Bluetooth connection from your VitalStim Plus device. The Software enables some additional options for your device, such as graph visualisation on a bigger screen, Patient's database, EMG Games, Template Training, Work/Rest Assessment and Training as well as many other useful functionality.



**Indication for use:**

The VitalStim Plus PC Software is an optional accessory to the VitalStim Plus device intended to display the sEMG measurements from VitalStim Plus for use in training exercises to aid in rehabilitation. The software receives information from VitalStim Plus, but does not influence the behavior of the VitalStim Plus device.

English


**Download**

Go to <http://www.djoglobal.com/vitalstim> to download the latest release.

**Install**

Run downloaded EXE file, your system may ask some security questions. Our files are tested against viruses, therefore click to allow the installation.

Follow the on-screen prompts. First, the software will install the necessary drivers and then you will be prompted to install the Software Application:

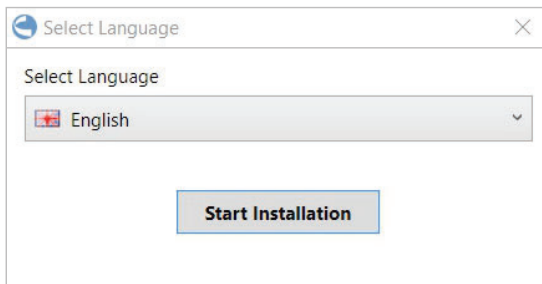

After the installation is completed, press Finish and launch the software.

**Supported systems:**

Windows Vista / 7 / 8 / 10. Windows 10 recommended!  
Avoid Windows XP as it is no longer supported by Microsoft, hence NeuroTrac Software is more in tune with the latest Windows platforms.  
Apple Mac, iPad, Windows RT are not supported. However, you can use Parallels on your Apple Mac to run this software.


**Hardware:**

Any new Windows PC or laptop with the standard or good graphical acceleration. The suggested system: Intel core i3 (or i5, i7 or more), min. 2G RAM, min. 500GB HDD, with the USB ports.

**Windows Tablets:**

Please note that cheaper Windows tablets have Windows RT system which is a mobile version of Windows and is not compatible with this PC Software as many crucial components are not there. The rule of the thumb is: Tablet with Windows 8 or 10 will always be compatible. For Example Microsoft Surface 1/2/3 platform will be compatible (It uses Windows 8 Pro).

Please fill in the registration details with up to date information. The registration details are stored locally and are printed on some of Software reports.

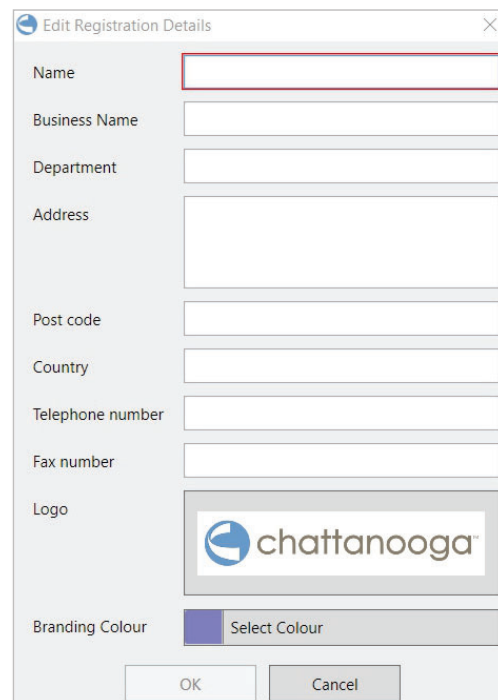

Finally, provide the new login and password.

Now you can login to your freshly installed version of Software. For more information please visit: <http://www.djoglobal.com/vitalstim>

### Upgrade from earlier versions

You may already have your valuable database with the data and logins from earlier versions. This installation will not harm your old installation, but will simply re-fresh the executable files and features leaving your data unchanged.

### Additional information and Troubleshooting

Please visit

<http://www.djoglobal.com/vitalstim> for the latest software manual, installation and troubleshooting.



DJO, LLC  
1430 Decision Street  
Vista, CA 92081 USA  
[www.DJOGlobal.com](http://www.DJOGlobal.com)



MDSS GmbH  
Schiffgraben 41  
30175 Hannover  
Germany