ENGLISH
BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

WARNING & PRECAUTIONS

Skin damage can result by placing HOT or COLD FlexiPac direty on skin! Place FlexiPac in FlexiPac Wrap, Therma-Wrap or toweling before use. Monitor condition of skin while in use.

Consult physician before use on skin desensitized or non-speaking patients.

Apply to children and infants with extreme caution.

Overheating may rupture FlexiPac packaging and cause burns! If packaging is damaged, dispose in accordance with local, state and federal regulations.

Gel contents are harmful if swallowed!

If swallowed, call physician and drink water or milk.

APPLICATION INSTRUCTIONS

FlexiPac for Cold therapy:

Place FlexiPac in freezer for at least one hour. Place FlexiPac in FlexiPac Wrap, Therma-Wrap or in toweling before applying to skin. Apply to treatment area for approximately 15 minutes. Return to freezer after use.

FlexiPac for Hot Therapy:

Microwave: Place FlexiPac in « microwave-safe » container with pack completely covered with water. Heat in microwave on high setting for approximately 1-2 minutes. DO NOT exceed 2 minutes for each use. Overheating may cause FlexiPac to rupture and leak. Place FlexiPac in FlexiPac Wrap, Therma-Wrap or in toweling before applying to skin. Apply to treatment area for approximately 20 minutes. Allow FlexiPac to return to room temperature after use and before re-heating.

Stovetop: Bring pot of water to boil. Remove water from heat. Place FlexiPac in water for 8-10 minutes. Place FlexiPac in FlexiPac Wrap, Therma-Wrap or in toweling before applying to skin. Apply to treatment area for approximately 20 minutes. Allow FlexiPac to return to room temperature after use and before re-heating.

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of one year from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2015 DJO, LLC
MDS GmbH